


DISTRIBUTED BY



# Disposable PCA Infusion Pump

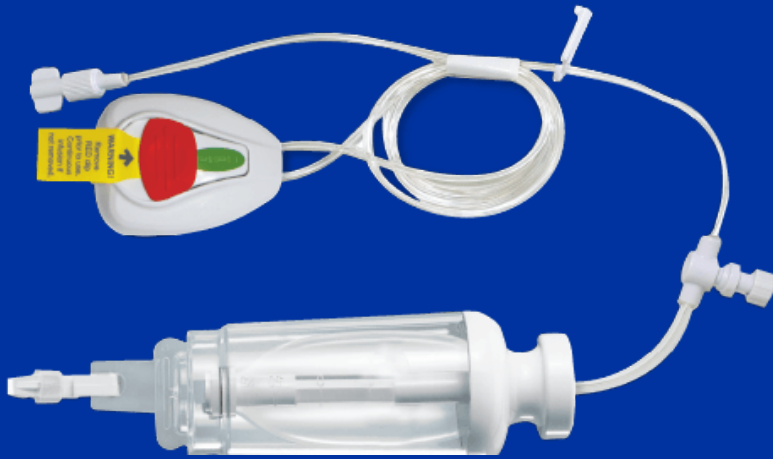

**Ariste Health's Disposable Infusion Pump is safe and effective.**

## **FULL DOSE INTERVAL (FDI)**

The elastomeric balloon will ensure a constant flow to the administration button reservoir.

This filling process takes 8 minutes. No further flow occurs unless the administration button is depressed, ensuring the patient receives only 1ml in any 8-minute period.

## **KEY FEATURES**

- **ACTIVE PRIMING**
- **NO POWER SUPPLY OR BATTERIES REQUIRED.**
- **ACCURATE FLOW RATE THROUGHOUT INFUSION.**

# Working principle

Ariste Health's Disposable Infusion Pumps feature Full Dose Interval and Active Priming. Driven by an elastomeric membrane, it offers accuracy and reliability.



## PCA Type

- ✓ Administers bolus dose on patient's demand
- ✓ Volume: 100ml
- ✓ PCA bolus volume: 1.0ml
- ✓ PCA FDI: 8min
- ✓ Suitable for postoperative pain management

## Basal Type

- ✓ Offer continuous infusion of medications at preset flow rates
- ✓ No need to program
- ✓ Volume: 100ml and 275ml
- ✓ Basal Flow rate : 10ml/hr
- ✓ Suitable for pain management, chemotherapy and antibiotic treatment

## Technical Information

- ✓ Detachable Particle Filter: Pore size 3µm
- ✓ Latex-free and DEHP-free
- ✓ Kink-resistant infusion tubing with triangle section
- ✓ EO-sterilized

### Product List

#### Product Code

- WZ-6523C-N
- WZ-6522-10
- WZ-6522-010

#### Nappi Code

- 485121001
- 489920001
- 489947001

#### Reservoir Capacity

- 100ml
- 100ml
- 275ml

#### Rate

- 0ml basal - 1 ml/8min FDI
- 10ml/hr
- 10ml/hr

#### Infusion Time

- PCA pump, 8min FDI
- Continuous Infusion
- Continuous Infusion

#### Pack Size

- 10
- 10
- 10